

Recommended cleaning solutions



The below lens cleaning solutions have been tested on our tinted soft contact lenses. At the time of creating this list, the solutions that passed our quality inspection are safe to use. Unfortunately, cleaning solution manufacturers can change the ingredients from time to time, which may affect the performance of the solution. If you have any information you would like to share with us regarding soft lens cleaning solutions please do not hesitate to contact us through our feedback form: -

<https://www.cantor-nissel.co.uk/contact-us>.

✓ - Recommended ✗ - Not recommended

Solution		Non opaque tint	Nissel custom	Cantor prosthetic
Alvera	multipurpose	✗	✗	✗
Complete Revitalens	multipurpose	✓	✓	✗
Eyedia	multipurpose	✓	✓	✗
Nautralens RDS	multipurpose	✓	✓	✗
Opti-free pure moist	multipurpose	✗	✗	✓
Opti-free Replenish	multipurpose	✓	✓	✓
Oté Sensation	multipurpose	✓	✓	✓
Oté Twins Active	multipurpose	✓	✓	✓
ReNu	multipurpose	✗	✗	✗
Avizor Lacrifresh Aloe	multipurpose	✓	✓	✓
Avizor All Clean Soft	multipurpose	✗	✗	✓
Avizor Unica Sensitive	multipurpose	✓	✓	✓
Kombilosung Aqua Plus	multipurpose	✗	✗	✗
Lensio Kombi-soft	multipurpose	✓	✓	✗
Menicare Soft	multipurpose	✓	✗	✓
Bio-True	multipurpose	✓	✓	✓
Oxi Care	peroxide	✓	✓	✗
Clear care	peroxide	✓	✓	✗
EasySept	peroxide	✓	✓	✗
Acuacare one step	peroxide	✓	✓	✗
Peroxi Clear	peroxide	✓	✓	✗

We are continuing to test different solutions. The list will be updated as the test results are completed.